



COMPLETE FORM



- 1 Complete a valid PreTRM order.



Contact PreTRM Customer Support for questions or additional information

(801) 990-6600 / support@pretrm.com


BLOOD DRAW



- 2 Have the patient wash their hands with warm water, rinsing thoroughly and fully dry hands.
- 3 Activate the hand warmer by flexing the metal disk; Place hand warmer on the finger to be used or on wrist for 30 seconds prior to collection.
- 4 Hold the finger in an upward position while holding the lancet to the palm-side surface of the finger. Use gauze to wipe off the first drop of blood.
- 5 Open the clamshell package containing the two Volumetric Absorptive Micro-Sampling (VAMS) devices.

- 6 Without removing the VAMS tips from the clamshell, using the first VAMS device, touch the tip to the blood; watch the tip turn FULLY red and then count 2 additional seconds. Repeat this process with the other sampling tip.

Ensure sampling tips are filled correctly. Under-sampling occurs when **1)** the sampling tip is removed from the blood too soon. Touch tip to blood until no white remains, or **2)** blood flow stops—if this happens, repeat steps 4-6 with same tip until it turns fully red.

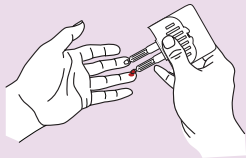

Always touch sampling tip to blood drop as shown at left

Over-sampled



Under-sampled



Correctly sampled



- 7 Close the clamshell device and place it back into the foil pouch containing the desiccant card.
- 8 Apply a bandage to the patient's finger.

PACKAGE & SHIP



- 9 Close the clamshell, and affix specimen label (with two patient identifiers and draw date/time) to the clamshell.
- 10 Place the clamshell back into the foil pouch containing the desiccant and seal, and place foil pouch back into the test kit box.
- 11 Place kit box into provided FedEx Clinical Pak envelope with shipping label on the outside of the FedEx Clinical Pak. Seal the FedEx Clinical Pak.
- 12 If you have routine FedEx pick-ups, provide the package to the FedEx courier. If you need to schedule a pick-up, please call (800) 463-3339 (800-GOFEDEX) and arrange for your pick-up using Sera Prognostics' account number 311458019.