
Kit contents

Note: if anything is missing or you have questions contact support@pretrm.com


First, unpack & prepare your kit contents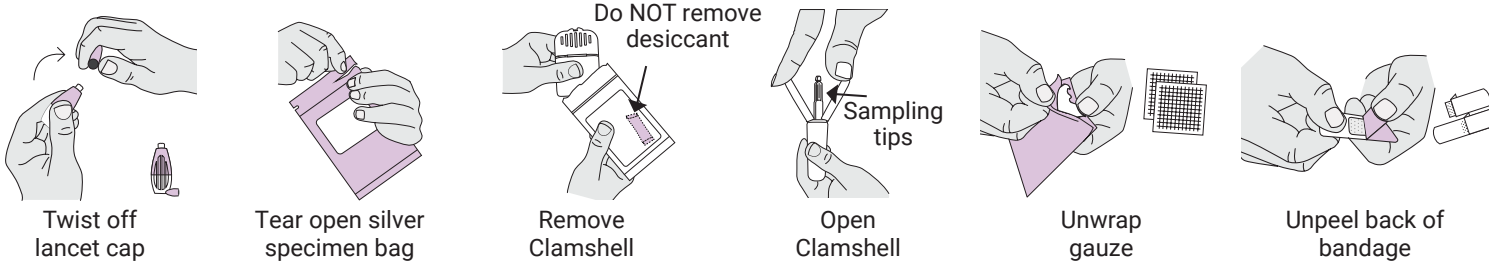

Instructions

- 1 Fill out the labels** to include your first and last name, date of birth, and the date and time that you collected your sample; this will be placed on Mitra Clamshell device after the collection.
- 2 Wash your hands** with warm, soapy water and fully dry hands.
- 3 Warm your hands** by flexing the metal disk of the hand warmer—place hand warmer on the finger to be used or on wrist for 30 seconds prior to collection. **Warming your hands is important for good blood flow.**
- 4 Select the finger you will use** to provide the blood sample. See illustration for recommended locations off-center of finger midline.
- 5 Prick your finger** by laying hand on a hard surface. Position the lancet on the chosen location. Apply firm pressure to end of lancet until a click is heard.
- 6 Wipe away first blood drop** with the gauze. A blood drop may take up to 20 seconds to form. If needed, gently massage pricked finger upward until drop forms.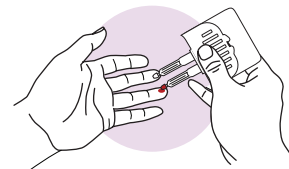

You may apply sampling tips to the blood drop multiple times to fill. Always point sampling tip towards floor when touching it to blood drop on your finger as shown above. **DO NOT drip blood onto the tip**

- 7 Collect sample by touching** one of the sampling tips to the blood drop. First, watch it turn fully red. Next, count 2 seconds. Last slowly remove the tip from blood. Repeat this process with the other sampling tip.

Ensure sampling tips are filled correctly. Under-sampling occurs when 1) the sampling tip is removed from the blood too soon. Touch tip to blood until no white remains, or 2) blood flow stops—if this happens, repeat steps 5-7 with same tip until it turns fully red.

Over-sampled


Under-sampled

Correctly sampled


Over-sampling occurs when blood is dripped onto the sampling tip from above. Always touch sampling tip to blood drop as shown in step 7.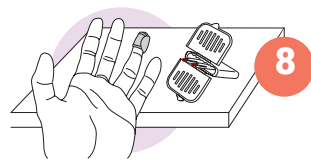


- 8 Set Clamshell device down** on hard surface and apply bandage to your finger.

- 9 Close Clamshell device** by lifting the sides to meet at the top. Press together until a click is heard. Add first label to the closed Clamshell device.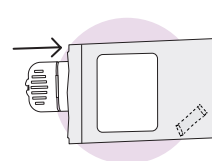

- 10 Insert Clamshell device** into the foil specimen bag and seal shut. Add second label to the outside of the foil specimen bag. **Ensure the desiccant is still in the bag.**

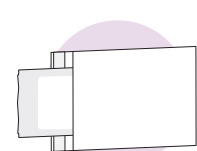- 11 Place kit box into provided FedEx Clinical Pak** envelope with shipping label on the outside of the FedEx Clinical Pak. Seal the FedEx Clinical Pak.

- 12 Ship to Sera.** You can drop it at any FedEx drop box, or you can contact Customer Support to request a pick-up.

You will receive an email notification when your results are ready to be accessed in the PreTRM Portal.

Contact PreTRM Customer Support for question or additional information at (801) 990-6600 or support@pretrm.com

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