You will receive an email notification when your result is ready to be accessed in the PreTRM Portal.

Contact PreTRM for Customer Support if you have questions or need additional information at (801) 990-6600 or support@sera.com

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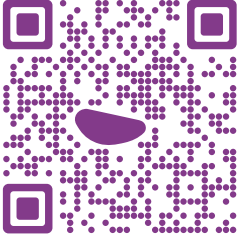
Only collect blood sample between **18w0d-20w6d** of your pregnancy

- Cautions:**
- For single use only. Do not reuse.
 - Do not use if packaging is damaged.
 - Keep out of reach of children.
 - Dispose of used materials properly

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BLOOD SAMPLE Collection Kit Instructions

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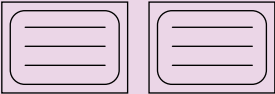
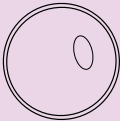
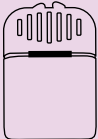
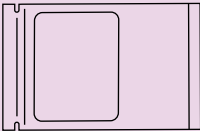
(1) Silver Specimen Bag
(1) Bolsa plateada para
la muestra

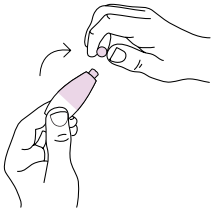
(1) Plastic Shipping Envelope
(1) Sobre plástico para envío


Note: if anything is missing or you have questions, contact support@sera.com

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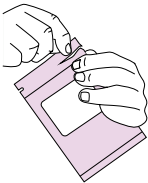
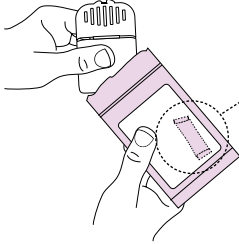
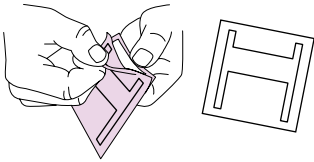
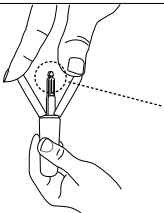
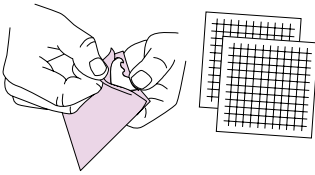
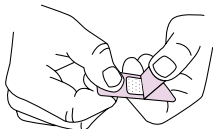
Ensure sampling tips are filled correctly. Under-sampling occurs when 1) the sampling tip is removed from the blood too soon. Touch tip to blood until no white remains, or 2) blood flow stops—if this happens, repeat steps 5-7 with same tip until it turns fully red.

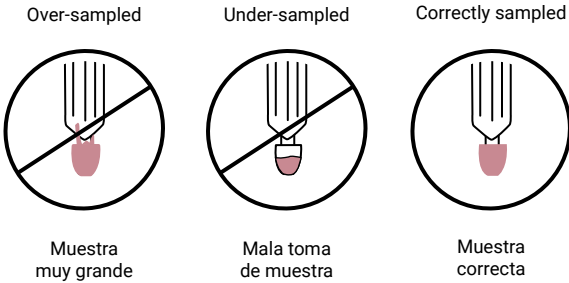Over-sampling occurs when blood is dripped onto the sampling tip from above. Always touch sampling tip to blood drop as shown in step 7.

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Instructions/Instrucciones

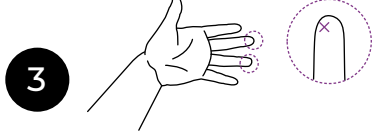
Fill out the labels using a pen to include your first and last name, date of birth, and the date and time that you collected your sample; this will be placed on the Mitra Clamshell device **after** the collection.

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Wash your hands with warm, soapy water and allow to air dry.

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
Select the finger you will use to provide the blood sample. See illustration for recommended locations off-center of finger midline.

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
Warm your hands by flexing the metal disk to activate the hand warmer—place hand warmer on the finger to be used for sample collection or on the wrist for 30 seconds prior to collection. **Warming your hands is important for good blood flow.**

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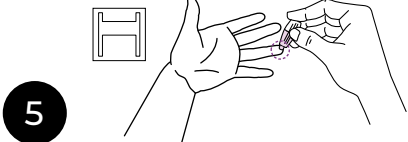
Disinfect your finger by using the alcohol prep pad to clean the fingertip. Next, prick your finger by laying hand on a hard surface with palm up prior to blood collection. Position the lancet on the chosen location. Apply firm pressure to end of lancet until a click is heard.

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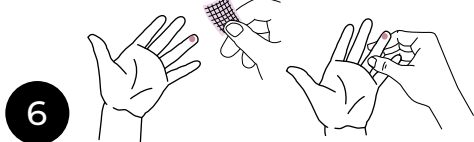
Wipe away first blood drop with the gauze. A blood drop may take up to 20 seconds to form. If needed, gently massage pricked finger upward until drop forms.

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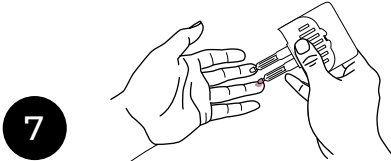
Collect sample by touching one of the sampling tips to the blood drop. First, watch it turn fully red. Next, count 2 seconds. Last slowly remove the tip from blood. Repeat this process with the other sampling tip.

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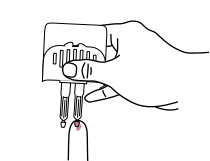
You may apply sampling tips to the blood drop multiple times to fill. Always point sampling tip towards floor when touching it to blood drop on your finger as shown above. **DO NOT drip blood onto the tip.**

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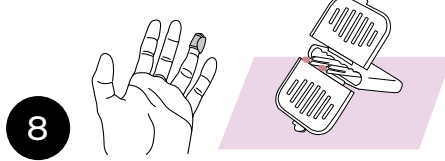
Set clamshell device down on hard surface with tips pointed up, and apply bandage to your finger.

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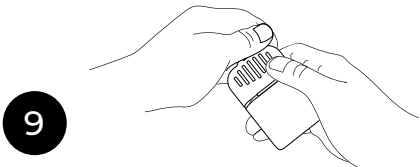
Close clamshell device by lifting the sides to meet at the top. Press together until a click is heard. Add first label to the closed clamshell device, leaving the side with the larger bar code still exposed.

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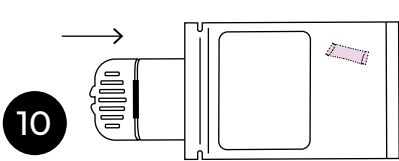Insert clamshell device into the foil specimen bag and ensure the desiccant is still in the bag. **Close the foil specimen bag by fully sealing the zipper.** Add second completed label to the outside of the foil specimen bag.

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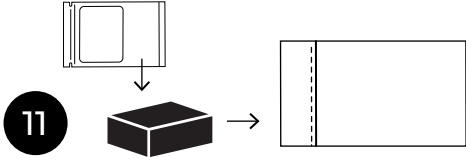Insert labeled foil specimen bag into the kit box. Place kit box into provided FedEx UN3373 Clinical Pak envelope with the prepaid shipping labels already placed on the outside. Seal the FedEx Clinical Pak by removing the liner to expose the adhesive.

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12 Ship to Sera. You can drop it at any FedEx drop box, or you can contact PreTRM Customer Support to request a pickup.*

*Dispose of Unused Material Properly

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